

# femaSeed<sup>®</sup>

## Procedure Guide

### Patient Preparation

- Perform pregnancy test, have patient void, and perform a bimanual exam
- Rule out infection requiring treatment and deferral of procedure

### Device Preparation


- Squeeze air from balloon and attach syringe filled with air
- Check balloon by inflating for 10 seconds
- Deflate balloon by pinching clips
- Leave syringe attached

*Air must be expelled from balloon to avoid damage.*

## 1 Fill FemaSeed with Washed Sperm

- Dispense prepared sperm into well of FemaSeed tray package
- Place tip of transfer catheter into well
- Pull back on built-in plunger to aspirate sperm into device

*Optional: If delivering sperm to both fallopian tubes, adjust marker for volume intended for each side.*

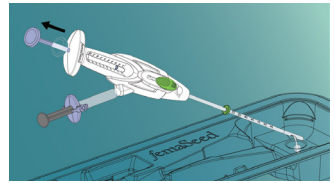


Fill FemaSeed

## 2 Place Device

- Place side-open speculum
- Retract transfer catheter by moving green slider back
- Under ultrasound guidance, insert FemaSeed to fundus

*Optional: Adjustable flange available if sounding is performed.*

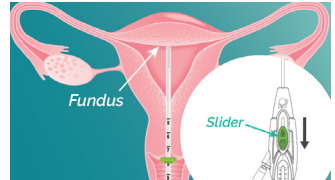

Device Placement


## 3 Advance Catheter

- Advance transfer catheter by moving green slider forward until clicks

*Note: StopLock syringe is located on the same side as the exit of the transfer catheter.*

- Slowly inflate balloon by depressing syringe piston

*Note: Transfer catheter must be fully advanced for balloon to inflate.*

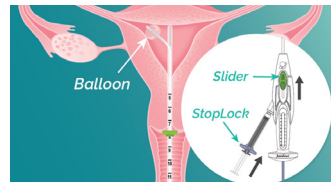

Catheter Placement

## 4 Deliver Sperm

- Using ultrasound, confirm balloon placement in cornu
- Deliver sperm by slowly advancing plunger

*Note: Marker available if delivering to contralateral tube.*



Sperm Delivery


## 5 Retract Catheter

- Deflate balloon by pinching clips of syringe
- Retract transfer catheter by moving slider back

## 6 Sperm Delivery Contralateral Tube (Optional)

- Repeat steps 3, 4, and 5 above

## 7 Remove Device & Discard

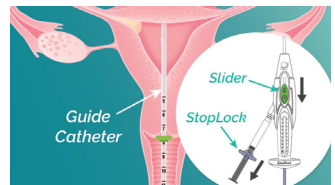

Catheter Retraction

## Ordering Information

To place a FemaSeed order, visit [femaseed.com/hcp](https://www.femaseed.com/hcp) or call **1-877-336-2562**